



Smart Technology, Made to Care

# Sit

Chair Occupancy Pad



Cair Sit is a chair occupancy pad which suits any type of chair. When used in conjunction with an occupancy sensor like the Cair Detect, it can alert immediately when a user leaves the chair, or after a period of time if they don't return.

The pad, which is made using soft vinyl to provide comfort for the user, can be placed above or below a chair cushion.

## Specifications

Compatibility	Cair Detect, and most 3rd party devices
Dimensions	L375 x W250
Length of Cable	1500mm
Type Of Plug	RJ12
Warranty	12 months

## Information

The Sit Chair Occupancy Pad can be used with the Cair Detect and most 3rd party devices. It should be placed above or below the chair cushion. Ensure the cable is tucked safely below or behind the cushion to avoid it becoming a trip risk.

When testing, follow the instructions for the connected device. Periodically test the pad to ensure it is functioning properly.

The pad should be cleaned regularly. To clean, use disinfectant wipes or apply some antibacterial solution to a damp cloth and wipe the sensor clean, ensuring moisture doesn't enter the area where the cable enters the pad. Allow to dry naturally or wipe with a clean, dry towel. Do not immerse the pad in any solution.

The pad should be replaced after one year of use. When in storage, keep the pad flat. Do not bend the pad.

