

Specifications

Compatibility	VB
Operating Frequency	869.2125 MHz
Dimensions	H115 x W69 x D28 mm
Input	2 x RJ12 inputs (some RJ11 connections are compatible - contact us for more information)
Selectable Options	Via easy to select rotary switch
Selectable Audio	Option for audio reassurance
Power Supply	AA batteries (User replaceable)
Battery Life	3 years under the following conditions - 5 alarms transmission per day
Wireless Range	Up to 600m line of sight
Compliance	CE / RoHS 2 / RED
Daily Presence Transmission	Every 24 hours
Warranty	24 months

Battery Replacement

Two 'Energizer L91 Ultimate Lithium AA' Batteries (Part no M115-110) should be used to ensure maximum battery life. It is important that the batteries are inserted correctly. Please note the polarity markings within the battery compartment. Once the batteries have been replaced, ensure the programmed settings are still correct.

Note: When replacing batteries, it is important to completely drain any residual power before new batteries are inserted. To do this, simply remove the old batteries, press and hold the on/off switch for 5 seconds, release the switch, then insert the new batteries.

Packing for shipment

The equipment containing cells or batteries must be packed in strong rigid packaging and must be secured against movement within the outer packaging to prevent accidental activation. The sender's name and return address must be clearly visible on the outer packaging.

Safety


Do not dismantle or alter the unit. Do not open the case. Indoor use in dry location only.

Disposal

All electronic waste should be disposed of in accordance with the latest legislation. It must be disposed of within the electrical and electronic waste stream and not be disposed of in the normal waste stream. Recycling electrical waste products help to conserve natural resources and prevent adverse effects on the environment. Contact your supplier should you require more information.

Declaration of Conformity:

Hereby, Cair (UK) declares that the radio equipment type, Cair Connect is in compliance with Directive 2014/53/EU

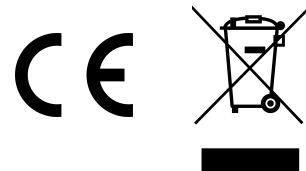

Cair (UK)

Hanson Lane, Halifax, West Yorkshire, HX1 4SD

T: 01422 399 155 E: sales@we-cair.com W: www.we-cair.com



Smart Technology, Made to Care

VB

Connect

Intelligent Universal Transmitter


Programming Guide v 1.0



The Cair Connect is an intelligent universal transmitter.

Simple and easy to use, the versatile Cair Connect can turn any hard wired device into a radio trigger. It works as an interface between hard wired devices and telecare enabled systems. Two RJ12 terminal inputs are provided for multiple device connections.

VB

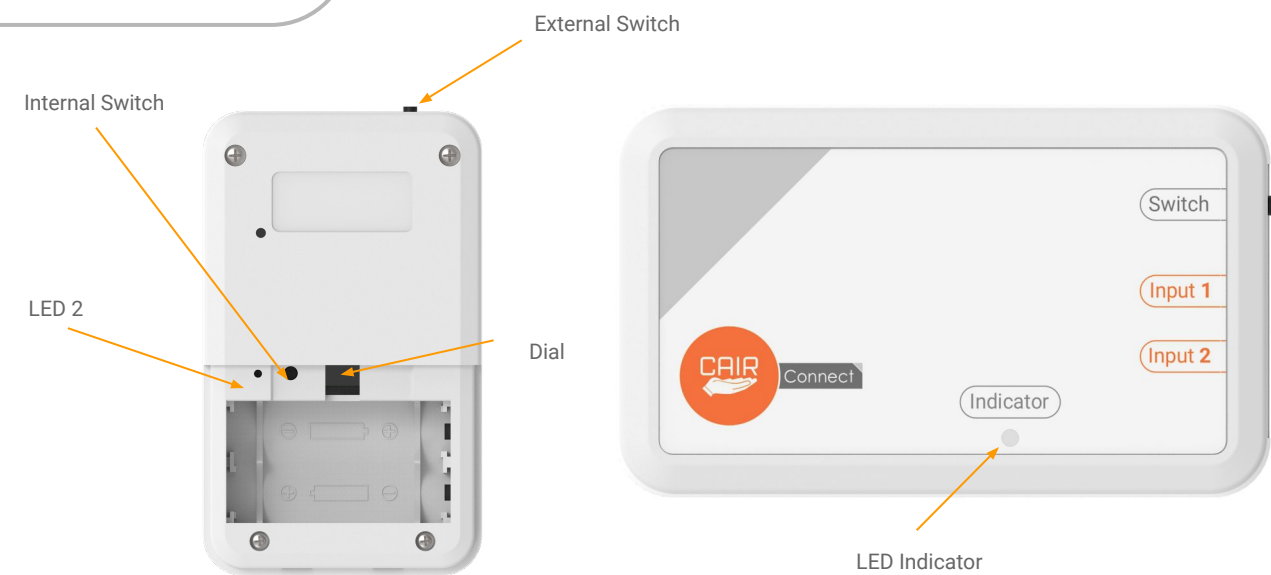

Switching On/Off

To switch on/off, press and hold the external switch. The LED Indicator will flash red and green briefly then produce a high-pitched tone followed by a brief green (on) or red (off) flash to confirm its current status. Now release the switch.

Operation

The Connect is an easy to use universal transmitter with 8 different device settings available. Two Identities are used; in all Modes Input 1 uses Identity 1 and Input 2 uses Identity 2. Please follow the instructions below and overleaf before attempting to program the Connect.

Assigning the Connect to a Notifier or other receiving system

1. Program the Connect to the desired Mode (if using modes 1-4 ensure the absence delay is set to a minimum).
2. Prepare the Notifier/other system by entering registration mode.
3. Activate the Connect.
4. The Notifier/other system should acknowledge the new device.

Programming the Modes/Options

To program the modes/options, use the following method:

1. Turn the dial to the desired setting
2. Press and hold the internal switch until LED 2 changes colour and a sound is heard (approx. 2 seconds)
3. Release the internal switch. LED 2 and the sounder will flash/beep to confirm the mode/option number you have chosen
4. Repeat steps 1-3 for other modes/options
5. If using Modes 1-4, simply set the dial to the desired absence delay as described overleaf

Contact Types for Mode 3 and 4

The default setting is for Normally Closed contacts. This means that when the door is closed, the contacts are closed and when the door opens, the contacts are open. It is also possible to programme to 'Normally Open', so that when the door is closed, the contacts are open and when the door is open, the contacts are closed. To programme the Connect to 'Normally Open' use the programming steps above, but press the switch once whilst the alternate red and green LEDs are flashing.

Absence Delay (Modes 1-4 only)

Once the modes/options have been chosen, simply turn the dial to the desired number which matches the delay time required. There is no need to press and hold the internal switch afterwards. The time settings are:

- | | |
|----------------|----------------|
| 0 – 0.5 Second | 5 – 1 Minute |
| 1 – 1 Second | 6 – 3 Minutes |
| 2 – 3 Seconds | 7 – 10 Minutes |
| 3 – 10 Seconds | 8 – 15 Minutes |
| 4 – 30 Seconds | 9 – 30 Minutes |

Modes/Options

The Connect is easily programmed using the dial positioned within the battery compartment. The dial is numbered 0-9. Each option is explained below:

- No.0.** Pendant Mode - Sends a radio message immediately on activation of the device.
- No.1.** Bed/Chair Absence Mode - Designed for use with Bed/Chair pads and works with the absence delays listed overleaf. This mode will send a radio message when the bed/chair is vacated. This mode will become active when weight has been applied to the bed/chair pad for more than 3 seconds.
- No.2.** Virtual Bed/Chair Mode - Designed for use with Bed/Chair pads and works with the absence delays listed overleaf. This mode will send a radio message when the bed/chair is entered and vacated. This mode will become active when weight has been applied to the bed/chair pad for more than 3 seconds.
- No.3.** Property Exit Sensor Mode - Designed for use with door contacts and works with the absence delays listed overleaf. This mode will send a radio message when the door opens. ('Normally Closed' or 'Normally Open' options can be used).
- No.4.** Virtual Door Mode - Designed for use with door contacts and works with the absence delays listed overleaf. This mode will send a radio message when the door opens and closes. ('Normally Closed' or 'Normally Open' options can be used).
- No.5.** Pressure Mat Mode - Sends a radio message immediately on activation of the mat.
- No.6.** Epilepsy Sensor Mode - Sends a radio message immediately on activation of the sensor.
- No.7.** Enuresis Sensor Mode - Sends a radio message immediately on activation of the sensor.
- No.8.** Sound Option - On/Off. A sound will be emitted when the Connect has activated (depending on chosen mode) and also when a transmission is sent. (Default - off)
- No.9.** Power Switch Option - Enabled/Disabled. When disabled, the Connect cannot be switched off using the external switch. To disable the switch (Default - Enabled).

Programming Example:

To program the following:

- No.1 (Bed Absence Mode)
 - No.8 (Sound On/Off)
 - Absence delay to 3 seconds (No.2 on dial)
1. Turn the dial to No.1
 2. Press and hold the internal switch using a suitable instrument until the unit beeps and LED 2 changes from red to amber (approx 2 seconds). LED2 and the sounder will flash/beep repeatedly to confirm the mode number you have chosen
 3. Turn the dial to No.8
 4. Press and hold the internal switch using a suitable instrument until the unit beeps and LED 2 changes from red to amber (approx 2 seconds). LED2 and the sounder will flash/beep to confirm the mode number you have chosen.
 5. Turn the dial to No.2 (3 seconds absence)